



# PIXEL

## INDOOR ROLLATOR

**PIXEL** is a multifunctional and very practical ultra-lightweight rollator for indoor use. It is for persons who want to maintain an independent lifestyle, but need a little help, support and safety to get around in the house. Besides stylish, unique and modern design, **Pixel** has a very low weight of only 5.2 kg.





### DESIGN

Stylish, unique and modern design. The frame of **Pixel** is made of aluminum.

### WEIGHT

With 5.2 kg (incl. tray table and textile basket) **Pixel** is one of the lightest indoor rollators on the market.

### COLORS

**Pixel** is available in 3 color combinations: classy black & white, natural toffee & brown and sophisticated black & champagne.



You also get a practical open net bag for storing personal items such as mobile phone, TV remote control, medicines, glasses, keys etc.



Includes a detachable tray in a nice smoke brown color, practical size and shape as standard. Dishwasher proof.




Includes a removable and washable textile basket for storage or moving of slightly bigger things as standard.



The hand brake control bar with user friendly paddles can be easily adjusted and works smoothly in two ways. It can be easily operated with one hand.




The push handle is easy to adjust to the required height (86-97 cm) with 5 increments that are marked with numbers 1-5. It can be also done with one hand.

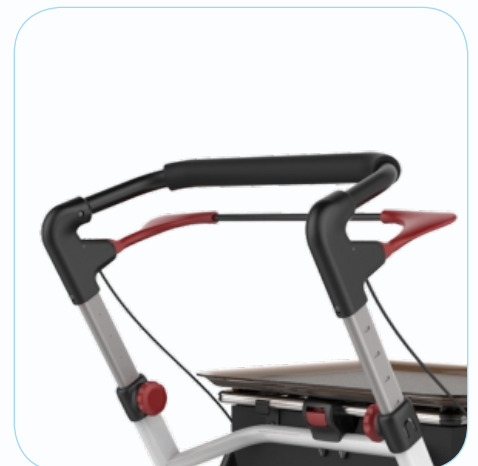

The hand grip is covered with soft foam and is very comfortable to hold.



REHASENSE

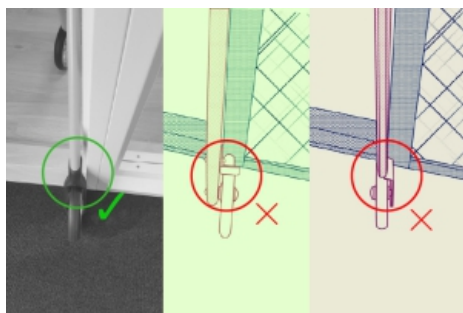

## EASY TO MANEUVER

Integrated rear wheels prevent the rollator gets stuck when passing through doorways, past furniture etc. and this prevents the interior from being scratched.



With its 54 cm width and swivelling front wheels, it can easily turn and pass very narrow doors, for example kitchen or toilet doors in old apartments.

## FOLDABLE

Convenient, space saving storage - **Pixel** is very easy to fold, unfold and store. Folding and unfolding can be done in 3steps. It stands on its wheels when folded. Remains locked in folded position. To unfold it, just do the opposite in reverse order.



## DINING TABLE & SIDE TABLE

**Pixel** and its tray can be used as a serving trolley or as a table to eat from.

The frame design allows **Pixel** to move closer to your chair, couch or bed, being used as side table.



Equipped with a unique locking system that makes it easy to lock or unlock the rollator when you want to fold or unfold it.



Wheels are equipped with soft, black PU tyres which provides excellent comfort and optimal grip on smooth surfaces/floors.



The rollator design combined with very low weight and super soft wheels provides extremely good performance when climbing door steps etc.

## ACCESSORIES

Crutch holder

Item number: **ACR01012P**



Anti slip mat for tray, black.

Item number **ACR01037**



Practical net bag with zipper

Item number: **ACR01037**



## FOLDING IN 3 STEPS



### Step 1:

Press the red button to unlock the basket and turn it to vertical position.

**Step 2:**  
Slide the basket down to the end as shown on the picture below.



### Step 3:

Fold the frame by pulling the basket towards you until you hear a light click sound.

## TECHNICAL SPECIFICATIONS

Item number:

White & Black

Toffee & Brown

Champagne & Black

PRWBL700

PRBRL700

PRCBL700

Total weight:

5,2 kg

Max. load:

110 kg

Push handle height  
(5 step heights):

86-97 cm

Total length:

64 cm

Total width:

54 cm

Dimensions folded  
(H x W x L):

91x54x27 cm

Tray dimensions:

43 x 33 cm

Tray height:

71 cm

Basket max load:

5 kg

**Pixel** comes fully assembled in a cardboard box ready for use.

Application: For indoor use.